Quick-Guide: MELAcontrol Type 5

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The indicator can be added to packaged instruments (e.g. in MELAfol, MELAstore) or placed directly on the tray in the chamber.

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Please start a sterilization program. The indicator is suitable for checking the sterilization result at 121 °C and 134 °C.

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When the sterilization program is finished, please remove the MELAcontrol Type 5 Indicator.

CAUTION: The load and indicator may be hot.

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After a successful color change from yellow to black, the color tone of the indicator no longer changes. This ensures complete documentation reliability.

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Document the result with date, name and batch number either directly on the test strip, on a separate documentation sheet or in a software solution.

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Note: The 5.3-minute holding time at 134 °C indicated on the indicator refers to a standard test cycle in a standard test device and is used for classification. Due to the higher control temperature of MELAG autoclaves, the envelope takes place at an earlier point in time. The indicator can therefore be used to check programmes with a holding time of 3.5 minutes.