

Checklist

Manufacturer's recommendations for the routine operation of sealing devices

Sealing device: _____ Serial number: _____

Daily check before operation:

Please make and check a sample seal seam as follows

Visual check					Mechanical check				Month / Year
The seal seam is distinct and even and does not contain any creases or defects.		The colour of the seal seam is identical with the factory-produced seal seam.			Peel test: Sterilize a foil packaging and open it afterwards. The paper should free itself from the film, leaving almost no residue.		Check of tensile strength The strength required to open the seal seam must be exactly equal to that required to open the factory-produced seal seam.		
Day	OK	not OK	OK	not OK	OK	not OK	OK	not OK	Signature:
1									
2									
3									
4									
5									
6									
7									
8									
9									
10									
11									
12									
13									
14									
15									
16									
17									
18									
19									
20									
21									
22									
23									
24									
25									
26									
27									
28									
29									
30									
31									

Weekly check:

Check the seal seam with MELAG Seal Check or MELAink Test.

Calender week		Calender week		Calender week		Calender week		Calender week	
OK	not OK	OK	not OK	OK	not OK	OK	not OK	OK	not OK
Signature:		Signature:		Signature:		Signature:		Signature:	

Monthly final check: _____
Signature